

● Deep

Deep dermis · cross-linked hyaluronic acid filler · class III · CE 2460

Device

Designation	Sofiderm Deep
Manufacturer	Hangzhou Techderm Biological Products Co., Ltd.
Classification	Class III, MDD 93/42/EEC, certificates in force under MDR art. 120 until 31-12-2027 (DNV letter C682153)
Notified body	2460 (DNV)
Exclusive distributor, Portugal	Orvida · Orquídeas Etéreas, Unipessoal LDA · NIPC 518912329

Composition and characteristics


Hyaluronic acid	20 ± 3 mg/ml, cross-linked (MCLPE: Multiphase Cross Linking & Purification Enhancement), in phosphate buffer, pH 6.8–7.6
Particle	0.28–0.5 mm
Gel	Dense gel, MCLPE cross-linking
Needle	25G and 27G, one of each in the 1 and 2 ml boxes
Formats	1 ml · 2 ml · 10 ml
Duration	6–24 months

Intended purpose (CE certificate)

Indication	Treatment of eyelid malposition.
------------	----------------------------------

Storage and use

Storage	2 to 30 °C, protected from light. Do not freeze.
Shelf life	Printed on the packaging; do not use after
Sterility	Steam sterilised; single use; do not re-sterilise; manufacturer needles only
Use	Licensed healthcare professionals trained in injectables only. The instructions for use limit the patient to 20 ml per 70 kg of body mass per year. Do not inject into the glabella.
Contraindications	Hypersensitivity to HA; active skin disease, inflammation or infection; autoimmune disease; bleeding disorders or anticoagulants; permanent implants at the site; not tested in pregnancy, breastfeeding or under 18. Full list in the IFU.

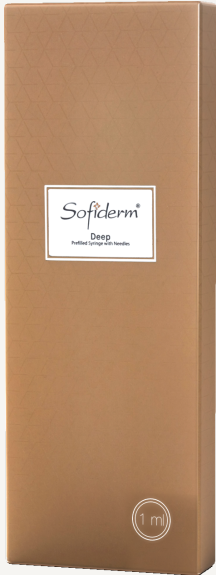

PROFESSIONAL PRICE · EXCLUDING VAT

€19 / 1 ml · €29 / 2 ml · €79 / 10 ml

Volume tiers in your account, after verification. No minimum quantity.

DOCUMENTATION ON REQUEST

CE certificate, DNV confirmation letter, declaration of conformity, manufacturer IFU.

